



1615 NE Eastgate Blvd, Sec H  
Pullman, WA 99163 USA  
Phone 509.592.8144  
[www.abt360llc.com](http://www.abt360llc.com)

## ABT PVA CONCENTRATE 10%

## ABT020, 20 ML

### General Information

Media should be gently mixed (not shaken) by inverting several times before use. Prior to use, **ensure solutions are clear and containers and ports are intact.**

**Recommended storage:** 2 – 30 °C. DO NOT FREEZE. For extended storage, refrigeration is recommended.

**Shelf Life:** 18 months from the date of manufacture.

### ABT PVA CONCENTRATE 10%

PVA's surfactant properties help prevent embryos from sticking to tubing, filters, or dishes after collection. This product is an alternative to BSA concentrate. It is packaged in a 20ml EVA bag and at 4ml/Liter, it is the same concentration of surfactant as ABT's Complete Flush.

**Recommended use:** Used as a surfactant when added to Lactated Ringers or Hartmann Saline Solution.

### *Disclaimer*

The recommended procedures in this insert are not considered a complete protocol and will vary according to user. ABT media are perishable products. Storage and handling will influence their rate of deterioration. ABT 360, LLC does not warrant any viability or conception resulting from the use of an ABT product. For further information contact us at (509) 592-8144 or visit our website at [www.abt360llc.com](http://www.abt360llc.com).